## Español BM PLT

### Para acoplar la botella/mochilaz

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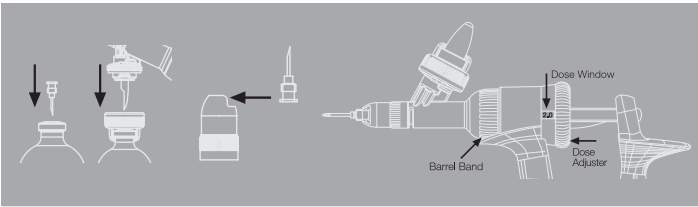
#### English BM PLT

##### To attach the bottle/pack

- 1 Pre-puncture the centre of pack bung with a sterile luer needle or equivalent (see above).
- 2 Unscrew & remove knurled clamp collar completely and remove draw-off spike cover.
- 3 Place pack neck into slot on clamp collar and then hold pack & collar with one hand, with pack in upright position.
- 4 Invert injector (i.e. bottle spike pointing downwards) and penetrate centre of rubber stopper with spike (see above).
- 5 Hold in place and tighten knurled clamp collar.

##### Note:

The Bottle Mount Injector is designed for use with both collapsible and non-collapsible vaccine packs and will therefore let air into the bottle/pack during use.

##### To fit needle

- 1 For safety - do not remove needle sheath until ready to prime.
- 2 Unscrew needle nut two complete turns. Do not remove from barrel.
- 3 Place rear flange of needle hub into the groove (see above).
- 4 Hold needle firmly in the groove and screw needle nut onto injector until tight. Do not remove needle nut.

##### To remove needle

Unscrew the needle nut in an anti-clockwise direction two complete turns. Slide the needle out of the open side of the needle nut.

Do not use pliers or excessive force as this may damage the unit.

##### Caution:

Please dispose of used needles in a safe manner.

##### Barrel Position

To optimise injector orientation the barrel band can be unscrewed and the barrel set to a different position if required. This is to ensure the bottle remains in an upright position during refill.

##### To prime

- 1 Ensure dose adjuster is set at the maximum dose in the dose window.
- 2 Hold the injector pointing upwards at an approximately 45° angle making sure the fluid level in the bottle is always above the draw-off spike (to avoid air being drawn into the injector).
- 3 Pump injector handle once and release quickly to draw vaccine into the barrel.
- 4 Pump handle forward again slowly until the air bubble has been discharged through the front of the barrel.

##### If the unit fails to refill during use:

- 1 Check that bottle is fitted correctly.
- 2 Disconnect the bottle and flush potential contaminants from valves by submerging injector in warm water and pumping vigorously.

##### Re-lubrication

If plunger becomes sluggish during use, remove the barrel by unscrewing and re-lubricate plunger O-ring with olive oil or any food grade vegetable oil.

##### Maintenance

Always flush the injector thoroughly with clean water immediately after use.

##### Sterilisation

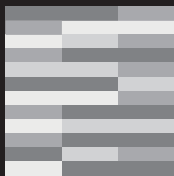
Use a cold water sterilant only.

**PARTNER.  
INNOVATE.  
DELIVER.**

Manufactured by:  
Simcro Limited  
6 Latham Court  
Hamilton 3204  
New Zealand

TEL +64 7 846 7723  
FAX +64 7 846 7738  
EMAIL [info@simcro.com](mailto:info@simcro.com)  
[www.simcro.com](http://www.simcro.com)

**SIMCRO™**

ITVGN093 Rev B

**SIMCRO™**

**SIMCRO™**

**PREMIUM INJECTOR**

**2ML FIXED DOSE | PLASTIC LUER-TIP**

Simcro is ISO 9001 Certified. All products are engineered for animal welfare, operator safety, treatment efficacy, and ease of use.